A.4080.1

A.4080.1.6

procto  plus

sapi med

SURGICAL
LINE

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*Adjustable, fenestrated, self-lighting anal retractor.
Disposable. Sterile.*

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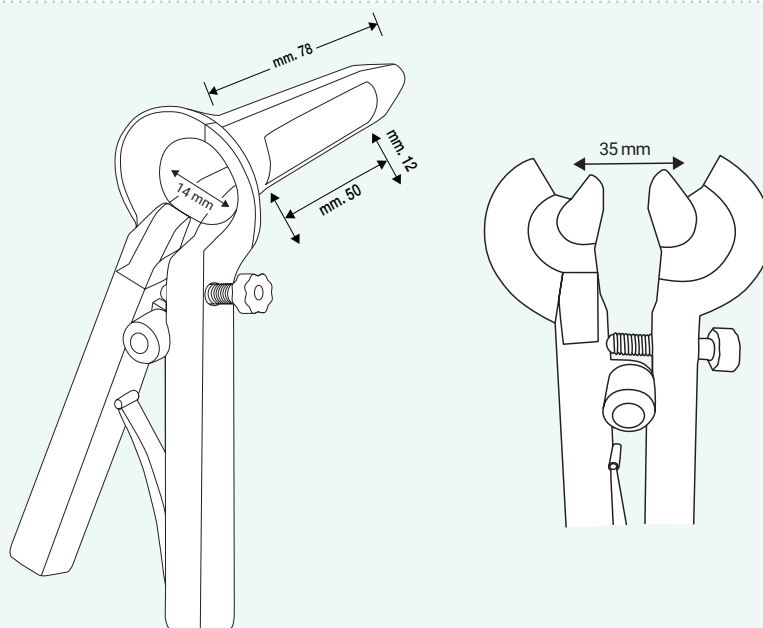
ProctoPlus is a versatile, single-use anal retractor that can be considered the evolution of the Eisenhammer retractor.

It consists of two blades, one of which is fenestrated, and two levers for a controlled opening.

A screw - positioned at the junction of the two levers - can lock the blades in the selected position.

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TECHNICAL FEATURES

- Sterile.
- Compatible with Sapi Med light sources.
- Perfectly transparent for 360° vision.
- Suitable for outpatient use due to the pediatric size.
- Atraumatic insertion.
- Adjustable and lockable blade opening.
- Maximum blade opening: 35 mm.
- Self-retaining for hands-free procedures in the operating room.
- Lateral fenestration for procedures on excess mucosa.

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The left lever is hollow to accommodate a light source. The light beam is magnified and directed precisely to the operating area. The large rectangular fenestration on the right blade allows for intervention on protruding mucosa.

Thanks to the materials used, ProctoPlus is very easy to handle and lightweight providing optimal vision.

When the blades are closed, they form a small, blunt-tipped cylinder, like a pediatric anoscope. The internal diameter of the ProctoPlus with closed blades is 14 mm, and it reaches a maximum distal opening of 35 mm.

This wide range in caliber allows for use in the outpatient setting for diagnostics or minor procedures requiring only topical or perianal anesthesia. It can also be used in the operating room for more invasive procedures under local or general anesthesia.

Thanks to the compact size of the blades, the surgeon avoids the risk of dragging the excess of mucosa into the anal canal upon insertion.

Once opened, the retractor enables clear assessment of any excess mucosa protruding from the lateral window or from the space created between the blades.

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Adjustable, fenestrated, self-lighting anal retractor. Disposable. Sterile.

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Adjustable, fenestrated, self-lighting anal retractor, disposable, sterile and 6 Assuofil 2/0 purple coloured suture threads, reinforced 27 mm needle 5/8 C. Disposable and sterile..

Consists of:

REF A.4080.1 (Anal retractor)

+ **REF FU429AT** (6 threads of suture) CE0373 - Manufacturer: Assut Europe S.p.A. - via G. Gregoraci,12 - 00173 Roma - Italy - Ph. +39 06 72677348 - customerservice@assuteurope.com

Sterilised by: ETO

Quantity per box: 6 pcs.